

EXIT-PAD®

AG

S I L V E R

Description:

Non adherent silver dressing.

Indications:

Dressing of the intravascular devices connection sites.
To be used for the compression and the absorption of exudates.

Purpose:

The silver layer has an immediate and long term antibacterial action helping to reduce the risk of local infection of the exit-site, of the tunnel or systemic (catheter-correlated sepsis) related to the use of intravascular devices. It helps to prevent the irritation caused by the mechanical rubbing of the catheter over the skin.

Besides, compared to other antiseptics, the silver ions of *EXIT PAD Ag* in a $<0,02\mu\text{g}/\text{cm}^2$ concentration, are well tolerated.

Sperimental evidences

The antibacterial action *EXIT-PAD-Ag* pads has been tested.


The results are shown in the down below diagram.

Antibacterial action of

Biochem test report n° 2498/08
Percentage reduction
of the starting bacterial concentration
incubation 0,5h- incubation 4h- incubation 24h

EXIT-PAD®

S I L V E R AG

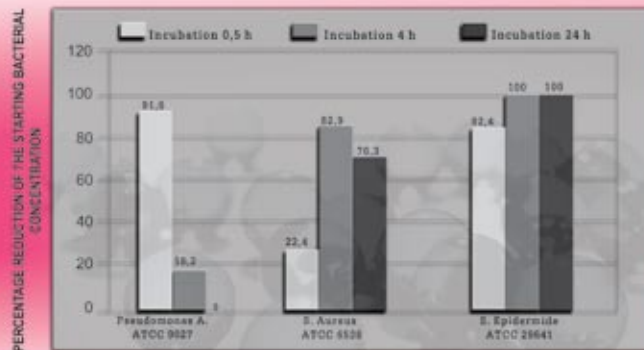

Method of the analysis

A standardised bacterial culture (minimum starting concentration of 1×10^8 cfu/ml) is placed onto the sample (pad with silver) and control (pad without silver). The count of the bacterial colonies grown is made immediately after sowing (0,5 h), after 24 h and 48 h incubation period. The results are expressed as percent bacterial reduction in *EXIT PAD Ag* sample vs. control.



Pat Pending



Direction for use

- 1 Put the pad over the catheter exit-site and hold in place with plasters.
- 2 Change dressing at every session.

REFERENCES Data and technical specification can be changed without notice

NON ADHERENT SILVER DRESSING

CODE	PAD Measure cm	Model	Quantity blister pack	Packages box
S0900/AG	3,5 x 4,5	WITHOUT CUT AND HOLE	1	250
S0903/AG	3,5 x 4,5	WITH CUT AND HOLE Ø 3		

Classification: Class III medical device
Biocompatibility: The material in contact with the skin has passed the biocompatibility tests according to UNI EN ISO 10993 standards
Sterilisation: γ - ray
Scadenza: 5 years